



## Sampling and Transport instructions for NOPHO Leukemia Biobank

### Sampling instructions:

- Bone marrow samples: 2-5 ml in 2 heparinized test tubes containing 2 ml 0.9 % saline
- Blood samples: 3-5 ml in 1 (if WBC<50 2 tubes) heparinized test tube
- Remission - day 71, blood sample: 3-5 ml in 1 (if WBC<50 2 tubes) EDTA test tube

### Sample time point:

- Diagnosis - first sample
- Relapse - at suspected relapse
- Other time points - all other sampling points
- Remission - day 71 for A2G (All Together protocol) for constitutional DNA in EDTA test tube.

### Transportation:

- Transportation has to be arranged and paid by each unit
- The blood- and bone marrow sample must be kept at room temperature
- Make sure that the sample is delivered to the lab in Uppsala on the following day, preferably before noon
- The blood- and bone marrow sample must be kept at room temperature
- Samples can be sent any day of the week
- For samples sent on Fridays, please send an e-mail to: [info@uppsalabiolab.se](mailto:info@uppsalabiolab.se)

***Referral form is to be filled in and sent with the samples.***

In case of difficulties, please contact UppsalaBioLab Phone: +46 (0) 18—617 23 88

Or send an e-mail: [info@uppsalabiolab.se](mailto:info@uppsalabiolab.se)

Send sample to:

Klinisk Kemi och Farmakologi  
Provinlämningen, Ingång 61, 2 trappor  
Akademiska sjukhuset  
SE-751 85 UPPSALA  
SWEDEN